



PRODUCT CODE: 256876

Paraffin Cleaner (CE-IVD) for clinical diagnostics

Microtomes cleaner used in the processing of human tissue, In Vitro Diagnostic medical device class A in compliance to the Regulation (EU) 2017/746.

TARIC 3822 90 00 00

PHYSICAL DATA: liquid, Clear, Colourless, Soluble in water 0,1 g/l at 20 °C D 20/4 0,766 • M.P.: -10 °C • B.P.: 96 °C • Flash P.:57 °C • Vap. press. 19 hPa •

INSTRUCTIONS FOR USE: First clean the microtome with a fine brush, removing paraffin not adhered to the instrument. Spray the apparatus part with paraffin spray cleaner. Wipe gently with a cloth or fine paper until the dissolved paraffin residue has been removed. Allow to dry.

BIBLIOGRAPHY:

HAZARDOUS: VLA-EC (1-propanol) 400 ppm1.000 mg/m3 VLA-ED (1-propanol) 200 ppm500 mg/m3



H: H226 • H318 • H304 • H413 •

P: P210 • P261 • P280 • P301+P310 • P303+P361+P353 • P305+P351+P338 • P310 • P331 • P403+P233 • P405 • P501 •

TRANSPORT REGULATIONS: UN: 1993 • ADR: 3/III • IMDG: 3/III • IATA: 3/III • PAX: 355 • CAO: 366 • (D/E) •
WEIGHT/VOLUME INFORMATION: 1l~0,766 kg 1kg~1,305 l

COMPOSITION:

Isoparaffin H	425 ml
1-Propanol	75 ml

SPECIFICATIONS:

Identity :
Identity

passes test

Maximum limit of impurities
Water (H₂O)

0,3 %